



# Multi-panel Drugs of Abuse Test

## DIPCARD

A rapid, one step screening test for the simultaneous, qualitative detection of multiple drugs and drug metabolites in human urine. It is intended for use by healthcare professionals including professionals at point of collection / care sites.

## Test Procedure

1. Collect urine specimen with urine container/cup
2. Remove a Dipcard test device from its original pouch.
3. Remove the cap from the test device. Hold the test device at a vertical position and dip it into the urine specimen for at least 20 seconds. (Do not allow urine to touch plastic card.)
4. Interpret the results in 5 - 20 minutes. Do not interpret results after 20 minutes.

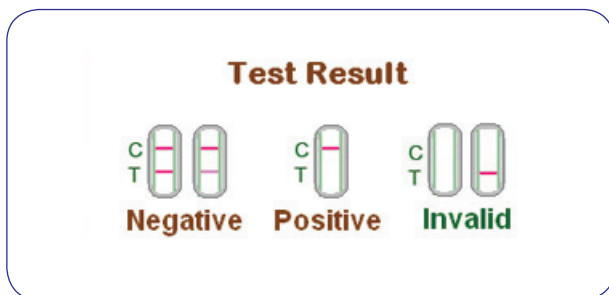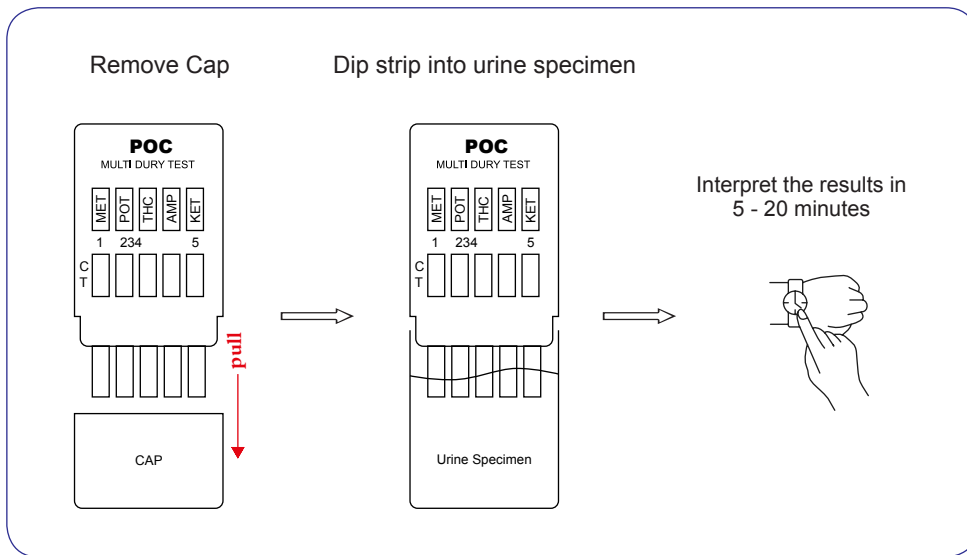

| Catalog Number | Description         | Packag   |
|----------------|---------------------|----------|
| POC21001       | 4mm X 2 test strips | 25 tests |
| POC31001       | 4mm X 3 test strips | 25 tests |
| POC41001       | 4mm X 4 test strips | 25 tests |
| POC51001       | 4mm X 5 test strips | 25 tests |
| POC61001       | 4mm X 6 test strips | 25 tests |